

# URISIL-LINE<sup>®</sup>

**URISIL-LINE** is an elastic fabric lined with medical grade silicone intended for the treatment of hypertrophic and keloid scars, or as prophylactic treatment on closed wounds to prevent the development of such scars. The silicone lined fabric may be embedded into pressure garments, and may also be used as separate finger garments.

## How to Care for your URISIL-LINE Product

Pressure garments with URISIL-LINE sewn in them should be washed daily in water with a non-greasy, mild soap and then rinsed in clean lukewarm water. The garment should be dried and reapplied to the clean, dry scarred area.

- ✓ Avoid cleaning the garment with detergents that might irritate the skin.
- ✓ Do not use paper towels to dry the fabric sheet, as they may leave particles that stick to the fabric sheet.
- ✓ Do not apply any cream to the scar when using **URISIL**.

Pressure garments with URISIL-LINE sewn in them should be used for a minimum of 12 hours per day throughout the treatment period. For best results, garment should be used for 23 hours per day and removed only for cleaning.

In order to allow scarred skin to properly adjust to treatment, pressure garments with URISIL-LINE sewn in them should be used for 2 hours per day for the first one or two days, 4 hours per day for the following two days and then increased incrementally 2 hours each day until the optimal treatment time of 23 hours per day is achieved.

When used to treat surgical scars, **URISIL** may only be applied following the removal of sutures.

Skin irritation on areas treated with silicone-lined fabric sheets may indicate either poor hygiene or excessive pressure when applying the fabric sheet. If a rash appears, discontinue the use of **URISIL** and consult your therapist.

**URISIL should not be used on open or secreting wounds.**

**Patients with dermatological problems should not use URISIL.**